# sigvaris

## Compreflex Transition

**INELASTIC WRAP - CALF**

INSTRUCTIONS FOR USE AND CARE

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# Compreflex Transition: Inelastic Wrap - Calf

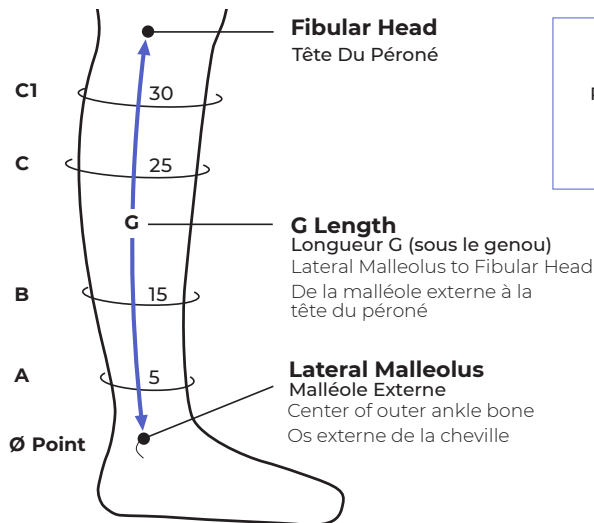
**Valid for** France, USA, Canada, Mexico, Brazil

## Product Measurement and Size Chart:

Identifying the optimal product size ensures a proper fit and the correct distribution of compression. Measurements should be taken by a healthcare professional.

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34–42cm

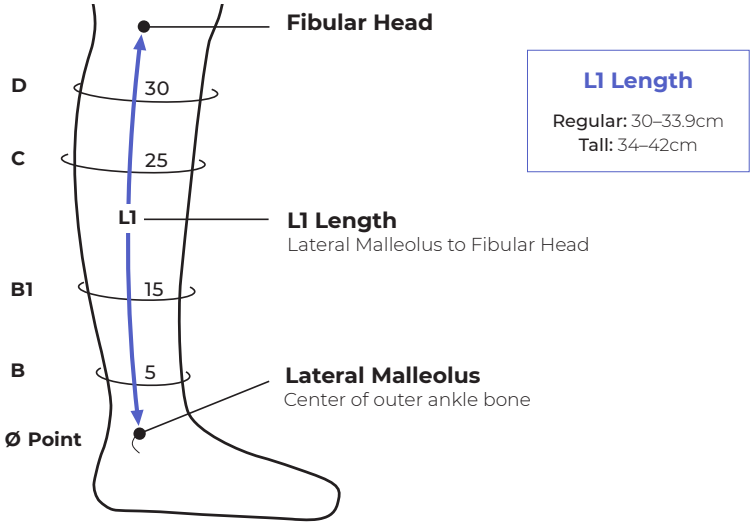
## Compreflex Transition Calf Sizing Chart (A–C1) Tableau De Tailage (cm)

	Small	Medium	Large	X Large	XX Large
C1	29–39cm	34–44cm	40–50cm	46–56cm	54–64cm
C	29–39cm	34–44cm	40–50cm	46–56cm	54–64cm
B	24–34cm	29–39cm	34–44cm	39–49cm	44–55cm
A	19–26cm	21–30cm	26–36cm	31–41cm	36–46cm

**Valid for** Rest of the World

**Product Measurement and Size Chart:**

Identifying the optimal product size ensures a proper fit and the correct distribution of compression. Measurements should be taken by a healthcare professional.

## Compreflex Transition Calf Sizing Chart (B–D)

	Small	Medium	Large	X Large	XX Large
D	29–39cm	34–44cm	40–50cm	46–56cm	54–64cm
C	29–39cm	34–44cm	40–50cm	46–56cm	54–64cm
B1	24–34cm	29–39cm	34–44cm	39–49cm	44–55cm
B	19–26cm	21–30cm	26–36cm	31–41cm	36–46cm

# Compreflex Transition:

## Inelastic Wrap - Calf

---

**MD**

en	<b>Medical Device</b>
fr	<i>Dispositif médical</i>
es	<i>Producto sanitario</i>
it	<i>Dispositivo medico</i>
de	<i>Medizinprodukt</i>
pt	<i>Dispositivo médico</i>
pl	<i>Wyrób medyczny</i>
nl	<i>Medisch hulpmiddel</i>
da	<i>Medicinsk udstyr</i>
fi	<i>Lääkinnällinen laite</i>
ru	<i>Медицинское изделие</i>
lt	<i>Medicinos priemonė</i>
sl	<i>Medicinski pripomoček</i>
tr	<i>Tıbbi cihaz</i>
ja	医療用製品
ko	의료 기기
ar	الجهاز الطبي

---

---

---

---

---

---

---

---

# Compreflex® Transition: Inelastic Wrap - Calf

---

## Product Descriptor:

The Compreflex Transition Calf is an inelastic compression device, providing accurate levels of compression of 20–50mmHg to the calf using the measurement device Accutab. The Accutab is also aligned to the RAL compression ranges of 21–46mmHg. This compression device is paired with the Transition Liner which provides protection for the skin and offers 10–15mmHg in the foot and ankle.

## Intended Use:

Inelastic wraps are compression therapy devices used on the upper and lower extremities. Compression therapy improves venous and lymphatic return, reducing venous and lymphatic edema. The product is intended for multiple use.

## Indications:

Lymphedema, Edema, Chronic Venous Disease (CVD), Chronic Venous Insufficiency (CVI) including, but not limited to: ulcer due to venous stasis, Lipedema

## Absolute Contraindications:

- Severe cardiac insufficiency (NYHA III-IV)
- Phlegmasia cerulea dolens
- Severe PAOD with any of the following: ABPI <0.6; ankle pressure <60mmHg; toe pressure <30mmHg
- Severe diabetic neuropathy with sensory loss or microangiopathy with the risk of skin necrosis
- Compression of an existing arterial bypass
- Allergic to compression material

## Precautions:

Special precautions must be taken, if the use of compression garments is considered in patients with border line indications such as:

- PAOD with ABPI >0.6 - <0.9 and/or ankle pressure between 60 and 90mmHg
- Patients with compensated heart failure (NYHA I-II) and venous or lymphatic edema
- Diabetes mellitus
- Skin diseases with exudation or mixed arterio-venous ulcer
- Compression garments for the reduction of inflammation, pain or edema of the leg, erysipelas or cellulitis should only be used in combination with antibacterial treatment
- Skin irritations in sensitive patients can be prevented by applying applicable skin care beneath compression garment
- Liners and wraps should not be worn directly on wounds

Treatment decisions should be taken on a case-by-case basis and under consideration of a careful benefit-risk assessment.

All serious incidents that occur when using the product should be reported to the manufacturer and the competent supervisory authority.

## Contents:

- 1 Compreflex Transition Calf
- 1 Pair of Transition Liners
- 1 Set of Accutab® Devices

## Material Composition:

- 39% Polyurethane foam
- 28% Polyester
- 24% Polyamide
- 5% Cotton
- 4% Elastane

**Care:**

- Close the velcro straps before washing
- To protect the garment, place in a mesh laundry bag for washing
- Machine wash on the gentle cycle using a mild detergent
- DO NOT use bleach or fabric softeners
- Tumble dry on low heat setting. Air drying is recommended to preserve the life of the garment
- Your Compreflex Transition Calf device should be washed as often as needed

**Caring for Your Liners:**

- Machine wash on gentle cycle using a mild detergent
- DO NOT use bleach or fabric softeners
- Tumble dry on low heat setting. Air drying is recommended to preserve the life of the garment

**Care & Cautions:**

Please read these instructions carefully and consult your medical professional if you have any health concerns. Your SIGVARIS GROUP compression garment should be comfortable and never hurt. If you experience any pain or discomfort, or visible marks on the skin indicative of skin reactions or constriction, immediately remove the garment and consult your physician.

**Application:**

1. Apply the provided liner. Before donning the Compreflex Transition Calf, unhook each strap and roll it back on itself to prevent tangling. Next, with the stretch panel in front, slide the garment on over the foot and pull up to just below the knee (garment tag will be at the back, facing out)
2. Beginning with the two straps nearest the ankle, gently lay the straps closed over the leg without applying compression, making sure there is minimal strap overlap and no gaps between straps. Press the velcro ends of the strap firmly onto the sides of the garment. Continue securing the remaining straps in the same manner
3. Apply the Accutab Devices using the instructions provided with the product. The Accutab instructions will also guide you in achieving the optimal level of compression
4. When doffing or removing the garment, roll the strap ends back to form a loop and press the velcro end of the strap onto itself to prevent the straps from tangling. Slide the garment off the leg, then remove the compression liner

**Use:**

While the Compreflex Transition Calf is suitable for day and nighttime wear, the garment should be removed once a day to allow for skin hygiene and wound care. It should then be reapplied according to the donning instructions. Frequency of garment removal for skin checks, hygiene, and/or wound care may vary according to each individual's needs and the prescribing provider's recommendations. Sigvaris recommends wearing a liner with each compression device.

..... **Not made with natural rubber latex** .....

**Note: Consult with your physician; precautions and contraindications listed are guidelines only and do not replace medical advice.**

# Compreflex Transition: Inelastic Wrap - Calf

---

## Product Descriptor:

The Compreflex Transition Calf is an inelastic compression device, providing accurate levels of compression of 20–50mmHg to the calf using the measurement device Accutab. The Accutab is also aligned to the RAL compression ranges of 21–46mmHg. This compression device is paired with the Transition Liner which provides protection for the skin and offers 10–15mmHg in the foot and ankle.

## Intended Use:

Inelastic wraps are compression therapy devices used on the upper and lower extremities. Compression therapy improves venous and lymphatic return, reducing venous and lymphatic oedema. The product is intended for multiple use.

## Indications:

Lymphoedema, Oedema, Chronic Venous Disease (CVD), Chronic Venous Insufficiency (CVI) including, but not limited to: ulcer due to venous stasis, Lipoedema

## Absolute Contraindications:

- Severe cardiac insufficiency (NYHA III-IV)
- Phlegmasia coerulea dolens
- Severe PAOD with any of the following: ABPI <0.6; ankle pressure <60mmHg; toe pressure <30mmHg
- Severe diabetic neuropathy with sensory loss or microangiopathy with the risk of skin necrosis
- Compression of an existing arterial bypass
- Allergic to compression material

## Precautions:

Special precautions must be taken, if the use of compression garments is considered in patients with border line indications such as:

- PAOD with ABPI >0.6 - <0.9 and/or ankle pressure between 60 and 90mmHg
- Patients with compensated heart failure (NYHA I-II) and venous or lymphatic oedema
- Diabetes mellitus
- Skin diseases with exudation or mixed arterio-venous ulcer
- Compression garments for the reduction of inflammation, pain or oedema of the leg, erysipelas or cellulitis should only be used in combination with antibacterial treatment
- Skin irritations in sensitive patients can be prevented by applying applicable skin care beneath compression garment
- Liners and wraps should not be worn directly on wounds

Treatment decisions should be taken on a case-by-case basis and under consideration of a careful benefit-risk assessment.

All serious incidents that occur when using the product should be reported to the manufacturer and the competent supervisory authority.

## Contents:

- 1 Compreflex Transition Calf
- 1 Pair of Transition Liners
- 1 Set of Accutab® Devices

## Material Composition:

- 39% Polyurethane foam
- 28% Polyester
- 24% Nylon
- 5% Cotton
- 4% Elastane

**Care:**

- Close the velcro straps before washing
- To protect the garment, place in a mesh laundry bag for washing
- Machine wash on the gentle cycle using a mild detergent
- DO NOT use bleach or fabric softeners
- Tumble dry on low heat setting. Air drying is recommended to preserve the life of the garment
- Your Compreflex Transition Calf device should be washed as often as needed

**Caring for Your Liners:**

- Machine wash on gentle cycle using a mild detergent
- DO NOT use bleach or fabric softeners
- Tumble dry on low heat setting. Air drying is recommended to preserve the life of the garment

**Care & Cautions:**

Please read these instructions carefully and consult your medical professional if you have any health concerns. Your SIGVARIS GROUP compression garment should be comfortable and never hurt. If you experience any pain or discomfort, or visible marks on the skin indicative of skin reactions or constriction, immediately remove the garment and consult your physician.

**Application:**

1. Apply the provided liner. Before donning the Compreflex Transition Calf, unhook each strap and roll it back on itself to prevent tangling. Next, with the stretch panel in front, slide the garment on over the foot and pull up to just below the knee (garment tag will be at the back, facing out)
2. Beginning with the two straps nearest the ankle, gently lay the straps closed over the leg without applying compression, making sure there is minimal strap overlap and no gaps between straps. Press the velcro ends of the strap firmly onto the sides of the garment. Continue securing the remaining straps in the same manner
3. Apply the Accutab Devices using the instructions provided with the product. The Accutab instructions will also guide you in achieving the optimal level of compression
4. When doffing or removing the garment, roll the strap ends back to form a loop and press the velcro end of the strap onto itself to prevent the straps from tangling. Slide the garment off the leg, then remove the compression liner

**Use:**

While the Compreflex Transition Calf is suitable for day and night time wear, the garment should be removed once a day to allow for skin hygiene and wound care. It should then be reapplied according to the donning instructions. Frequency of garment removal for skin checks, hygiene, and/or wound care may vary according to each individual's needs and the prescribing provider's recommendations. Sigvaris recommends wearing a liner with each compression device.

..... **Not made with natural rubber latex** .....

**Note: Consult with your physician; precautions and contraindications listed are guidelines only and do not replace medical advice.**

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® Sigvaris and SIGVARIS GROUP are registered trademarks of SIGVARIS AG, CH-9014,
St. Gallen/Switzerland, in many countries worldwide. © 2021 Copyright SIGVARIS, Inc.
SIGVARIS AG
SIGVARIS GmbH